

EZee Tape Information Sheet



Overview:

EZee Tape kinesiology tape is an elastic, therapeutic tape that can be beneficial to a wide range of injuries, pain reduction and physical support. It is made of high quality cotton with an acrylic, adhesive coating. EZee Tape kinesiology tape technology pulls the upper layer of the skin to create more space between the skin and muscles. This promotes the blood flow and lymphatic drainage.

Size:

Width: 5 cm (2")
Length: 6 m (19.6')

Product Numbers:

Colour/Style	Synthetic	Cotton
Beige	CH5300	CH5320
Blue	CH5301	CH5321
Pink	CH5302	CH5322
Black	CH5303	CH5323



EZee Tape's major functionalities are:

- Support of muscles and joints
- Reduction of muscle fatigue
- Stimulation of the lymphatic drainage and blood flow
- Reduction of swelling and pain
- Stimulation of the natural healing process.

Storage information:

- 1) EZee Tape kinesiology tape should be stored at approximately (21°C). High or low temperatures may degrade the quality.
- 2) Protect EZee Tape kinesiology tape from direct sunlight, sources of heat or humid locations.

Tips and information for use

- 1) Prior to usage, we recommended testing a small piece of EZee Tape kinesiology tape. In case of allergic reactions, oversensitivity or irritations, immediately remove and discontinue use of the tape. Notice that some reactions are not immediately perceptible. Therefore it is advised to test approximately 48 hours before use.
- 2) Prior to usage, the skin needs to be cleaned and dried for optimal adhesion.
- 3) Apply EZee Tape kinesiology tape with a small amount of stretch onto the skin. The ends of the tape need to be free of stretching. If the stretch is too high, remove the tape.
- 4) It is recommended to remove EZee Tape kinesiology tape in hair direction. Prevent the skin from high tension when removing the tape.
- 5) Round all corners of the tape pieces for optimal adhesion.

Warnings

- 1) EZee Tape is only to be used under the direction of professional therapists or specialists with adequate level of knowledge in the field of kinesi taping.
- 2) EZee Tape should only be used on healthy skin.
- 3) EZee Tape kinesiology tape may not be used on the face, damaged skin, burns or irritated skin.
- 4) Keep EZee Tape kinesiology tape out of reach of children.

EZee Tape kinesiology tape supports the natural healing process and cannot be considered as medication!

EZee Tape kinesiology tape does not guarantee injury prevention! EZee Tape cannot be held liable for such injuries.

All liability is precluded if the recommendations for use are not taken into account.

For Application Information & Further Instructions please visit our website at: <http://www.factorydirectmedical.com/EZee-Tape>

Imported / Distributed by:

Factory Direct Medical
34 Futurity Gate, Unit # 15, Concord Ontario L4K 1S6
Local: 416-739-8393 Toll Free: 855-235-2400

